

Registering with myAir is easy. Let's get started.

You can register either online at **myAir.ResMed.com** or through the myAir app, available for your Apple® iPhone® on the App Store® or for your compatible Android™ smartphone on the Google Play™ store.



Scan me!



Get connected with myAir



Register on any web-accessible device
at **myAir.ResMed.com**



myAir™



Get started with ResMed myAir™


Start tracking your therapy from your very first night with ResMed myAir.™ Available with any ResMed Air10™ machine, myAir is an online support program and app* that's like having your own personal sleep coach.

* The myAir app is available in English only and in the US, Canada, Australia and New Zealand only.
Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll free). See ResMed.com for other ResMed locations worldwide. Air10 and myAir are trademarks and/or registered trademarks of the ResMed family of companies. Android and Google Play are trademarks of Google, Inc., registered in the U.S. and other countries. Apple, App Store and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. Specifications may change without notice. For patent and other intellectual property information, see ResMed.com/ip.
© 2019 ResMed. 10111223/1 2019-07

myAir.ResMed.com

myAir.ResMed.com

Quick and easy myAir registration

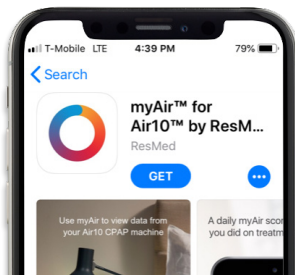


Step 1: Get started

Go to the app store on your phone, search for “myAir” and download the app. If you are registering online, visit: myAir.ResMed.com



Scan me!

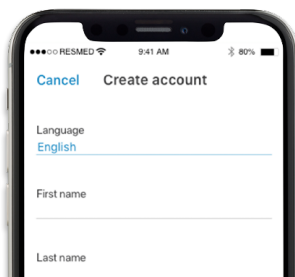


Step 2: Create your account

Type in your name, email address and all other information required on the “Create account” page. **Write down your login information – you’ll need it to finish registering.**

Email address:


Password:

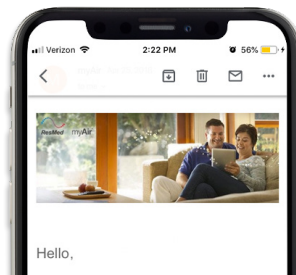

Step 3: Verify your account

Go to your email inbox and look for an email from “myAir” with the subject line “Please activate your myAir account.” Click the link in the email to activate your myAir account.

Follow the prompts on the screen to enter your email address and password.

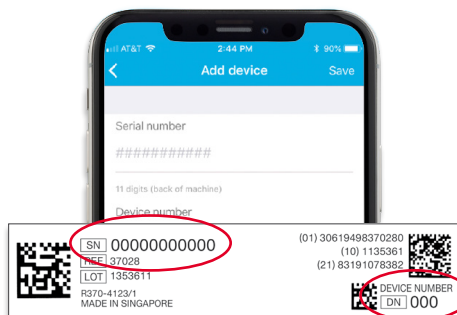

Step 4: Add your equipment

If you’ve already received your Air10 machine, follow the prompts on the screen to enter your machine’s serial and device numbers. You will also be asked to enter information about your mask. If you haven’t received your machine and mask yet, you can complete your registration after you’ve received your CPAP equipment.



Step 5: Start tracking your progress

You’ve successfully completed your myAir registration! Now you’re ready to track your sleep therapy. Be sure to check your myAir score each morning to see your progress.

myAir™